




# How to power down a Hamilton C2

This technique shows how to power down a Hamilton C2 ventilator.

Written By: knipwim



## Step 1 — shut down the ventilator

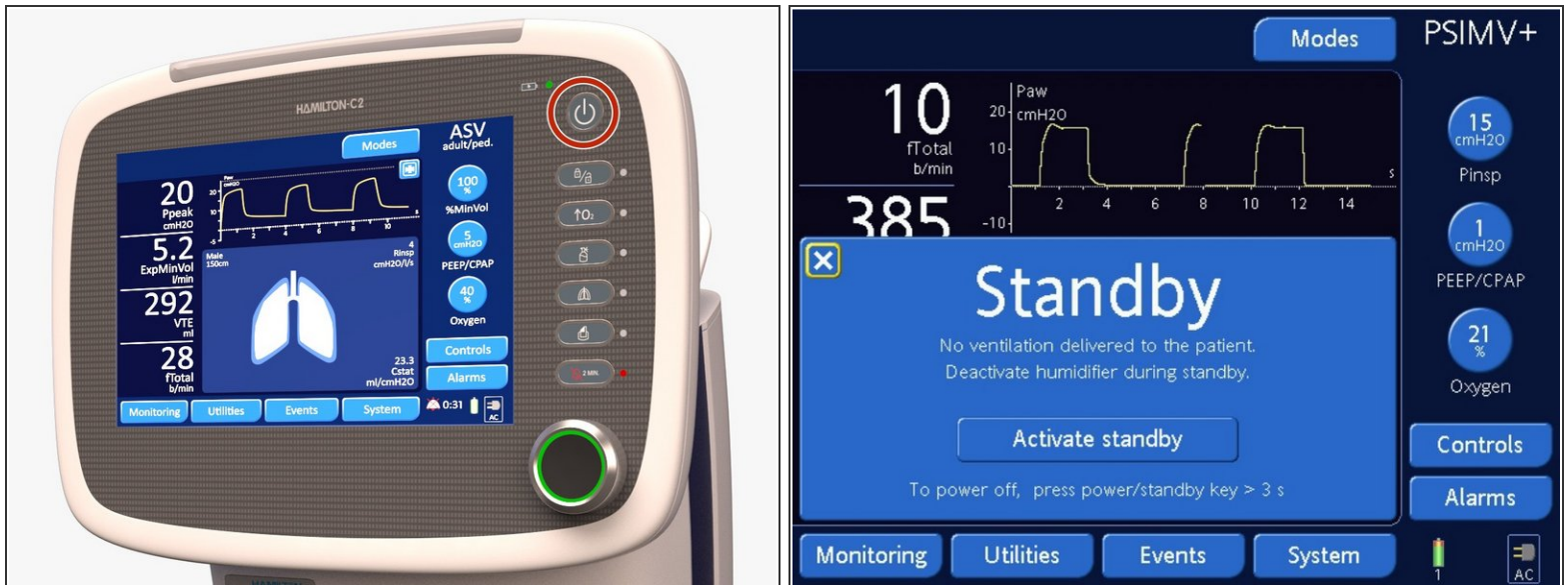

- Press and quickly release the power switch to open standby window in the screen.
- Activate the standby mode by pressing the **Active standby** button in the screen.
- Press and hold the switch for greater than 3 seconds to shut down the ventilator.
- ❗ If there is a technical fault, press and hold the switch for greater than 10 seconds.

## Step 2 — disconnect from power



- Disconnect the ventilator from the AC or DC power.