




# How to power down a Newport HT70

This technique summarizes how to power down a Newport HT70 ventilator.

Written By: knipwim



## Step 1 — shut down the ventilator

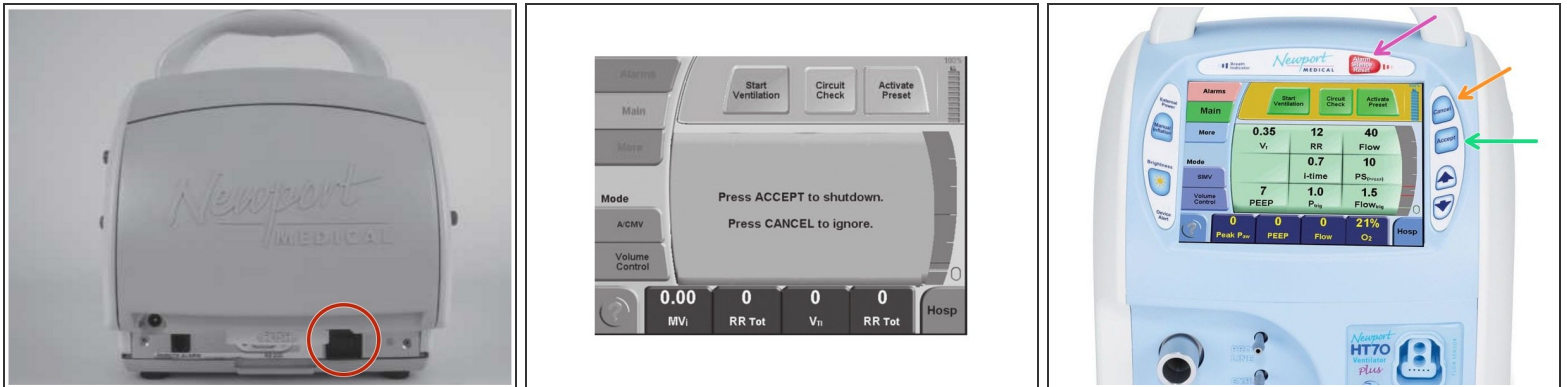

- The momentary-type power switch is located on the left side rear of the ventilator along the bottom edge. Press the power switch once.
- A message appears in the screen that prompts you to **Press Accept to Shutdown** or **Press Cancel to Ignore**.
- Pressing the **Accept** button will power off the ventilator.
- Pressing **Cancel** will return the ventilator to its previous state.
- Press **Alarm Silence** to silence the Shut Down alarm.

## Step 2 — disconnect from power



- Disconnect the ventilator from the AC power.