



How to power down a Philips Respironics V60

This technique summarizes how to power down a Philips Respironics V60 ventilator.

Written By: knipwim



Step 1 — shut down the ventilator



- Press and release the **ON/Shutdown** key, opening the **Shutdown** window in the screen.
 - Select **Ventilator Shutdown**, which will shut down the ventilator.
- ⚠ Improper shutdown may cause a **Power has been restored** message the next time the ventilator is turned on.

Step 2 — disconnect from power



- Disconnect the ventilator from the AC power.