



Reattaching the GEN 2 Wheelchair Foot Support

Occasionally the foot supports on the wheelchair will fall off because of normal wear over time. They are usually not broken and just need to be reconnected.

Written By: Kali Slaughter



INTRODUCTION

This guide will show how to reconnect the foot support to the wheelchair after it has fallen off.



TOOLS:

- [10mm Wrench](#) (1)
-

Step 1 — Foot Support




- Slide the footrest sockets down as shown and use a 10 mm wrench to tighten the bolts at the desired height.

Step 2



- Screw in the bolt to the top socket to secure the height of the footrest.

Step 3

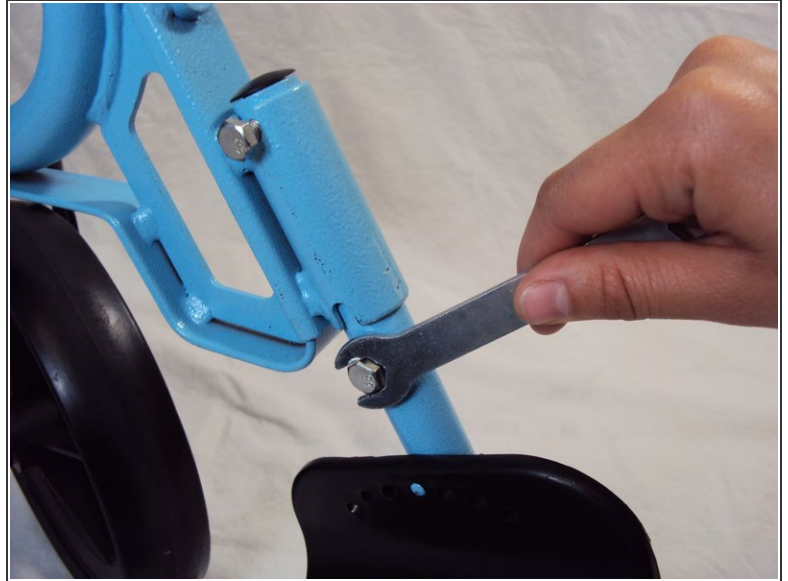

- Screw in the bolt attaching the footrest to the frame.
- Screw in the lower bolt to the footrest.

Check that the footrest is at the desired height. If the footrest is still uncomfortable start the guide over again from step 1 using a different height to tighten bolts.