



# Welch Allyn Connex Vital Signs Monitor 6000 Stand Replacement

Remove your Connex vital signs monitor from its office cart, or transfer it to a new post mount on a new stand.

Written By: Sam Goldheart



## INTRODUCTION

Use this guide to remove the Connex Vital Signs Monitor from the Welch Allen office cart stand.

PLEASE NOTE: This repair guide was developed by the iFixit team based on Welch Allyn's own service manual. Neither iFixit nor this repair guide is endorsed by or affiliated with Welch Allyn.

---

### TOOLS:

- [Flathead Screwdriver](#) (1)
  - [Phillips #2 Screwdriver](#) (1)
-

## Step 1 — Stand



- Unplug the AC power connection cable.

## Step 2



- Place your thumb and forefinger on the blood pressure (NIBP) hose connector. Squeeze the side tabs until the connector releases.
- ⚠ Always grasp the hose by the connector. Do not pull on the hose itself.
- Pull the connector away from the connector port.

### Step 3



- Place your thumb and forefinger on the Pulse oximetry (SpO2 or combined SpO2/SpHb) cable connector. Squeeze the side tabs until the connector releases.  
 Always grasp the cable by the connector. Do not pull on the cable itself.
- Pull the connector away from the connector port.

### Step 4



- Grasp the temperature probe and pull it up to remove it from the monitor.

## Step 5



- Remove the cover of the temperature module by pressing the tab and sliding the cover to the right.

## Step 6



- Depress the spring tab on the temperature probe cable connector and withdraw it from the probe port.

## Step 7



- Remove the flathead screw on the USB networking door.
- Loosen the captive Phillips #2 screw securing the monitor to the stand.

## Step 8



- Holding the monitor securely, open the USB networking door.
- ⓘ You may have to lift the monitor slightly to allow the door to swing out from the stand mount.

## Step 9



- Detach any accessory USB cables from USB ports on the monitor.

## Step 10



- Slide the monitor off of the stand tray.

To reassemble your device, follow these instructions in reverse order.